



LiquiBand® Flow Control

The ideal solution for rapid polymerization and superior control.



✓ **ENHANCED PATIENT BENEFITS⁵**

- ▶ Water resistant barrier allowing patients to shower after use
- ▶ Low exothermic reaction for increased comfort



✓ **FLOW CONTROL APPLICATOR**

- ▶ Provides a controlled and continuous flow of adhesive

✓ **100% BUTYL FORMULATION**

- ▶ Fast set time²
- ▶ Strong, secure wound closure
- ▶ Low viscosity for even application



Application Guide *LiquiBand® Flow Control*

✓ STEP ONE

- ▶ Ensure that wound is thoroughly clean and dry prior to closure.
- ▶ While holding upright, twist and snap off the winged cap.
- ▶ Affix the applicator tip with gentle pressure (optional).



✓ STEP TWO

- ▶ Invert the tip and gently squeeze to prime the device. When the tip fills with adhesive, LiquiBand® Flow Control is ready to use.




✓ STEP THREE

- ▶ While holding the wound edges together, apply an even coat of LiquiBand® Flow Control over the length of the wound.
- ▶ Continue to hold the wound edges together for 30 seconds.



LiquiBand® Flow Control

Size	Qty Per Box	Product Code	NHS SC Code
0.5g	10	LBFC 0010	FVF052

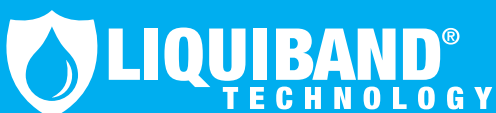

T +44 (0) 1752 209 955
E info@liquiband.com
W liquiband.com