



SINGLE USE



Instructions for use


LiNA Port

Single use 5mm Trocars

**REF: 1005-1, 1005-2, 1007-1, 1007-2, 1007-2-2GP
1008-1, 1008-2, 1009-1, 1009-2, 1009-C-1, 1009-C-2GP, 2005-1,
2005-2, 2007-1, 2007-2, 2008-1, 2008-2, 2009-1, 2009-2**

Usage:

- 1) Open pouch with the trocar and cannula.
Place the trocar and cannula on a sterile field.
- 2) Introduce the trocar into the cannula.
- 3) Choose the introduction site according to laparoscopic techniques.
- 4) Push the trocar through fascia while frequently watching the monitor.
- 5) Remove the trocar from the cannula as soon as the trocar is visible in the abdomen.
- 6) Turn the cannula clockwise in order to secure it in fascia.
- 7) After the procedure remove the cannula by turning the cannula counter-clockwise and pull it out.

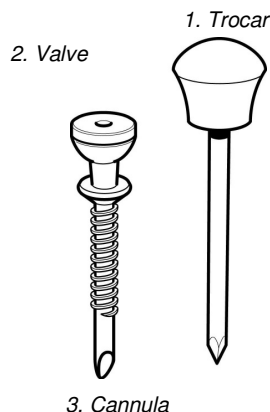

If you have further questions please do not hesitate to contact us:



NF EN ISO 13485
ISO 9001

LiNA Medical ApS
Formervangen 5
DK-2600 Glostrup
DENMARK
Tel. +45 43 29 66 66 - Fax +45 43 29 66 99
Email: linamed@lina-medical.com
www.lina-medical.com

FV0003B