



LiNA Veress Needle

*120 and 150mm single use insufflation
needle for laparoscopic procedures*



- *Ergonomically designed handle for comfortable gripping*
- *Blunt tip - spring-activated stylet with an audible click and a visual abdominal free-space indicator*
- *120mm or 150mm length*
- *Single use - eliminates reprocessing of hard-to-clean reusables*

LiNA Veress Needle

Patient safety as well as product quality and functionality has been the core focus of the design of the LiNA Veress Needle.

The needle cannula is available in 120mm or 150mm length. Inside the cannula and extending beyond the sharp tip is a spring-loaded, blunt stylet. The stylet retracts as the needle is pushed through the abdominal wall and automatically slides forward once the peritoneum has been penetrated. An audible click and a visual abdominal free-space indicator verify the correct position in the abdominal cavity.

The luer lock with a stopcock at the end of the handle is connected to the CO2 insufflation tubing for inflating the abdominal cavity.

LiNA Veress Needle 120mm Ref. 6650-120



Extending beyond the sharp cannula tip is a spring-loaded, blunt stylet to reduce the risk of inadvertent tissue damage.

LiNA Veress Needle 150mm Ref. 6650-150

Ordering Information:

LiNA Veress Needle **CE** 0459

REF.	Product	Packing
6650-120	LiNA Veress Needle - Insufflation needle for laparoscopic procedures. Length 120mm. Single use.	Box of 25, sterile
6650-150	LiNA Veress Needle - Insufflation needle for laparoscopic procedures. Length 150mm. Single use.	Box of 25, sterile

LiNA distributor in your country:

